

AVAILABLE TO DIGITAL CUSTOMERS ONLY

Argen Clear Aligners



HOW IT WORKS

The Argen Clear Aligner kit comes complete with as many stages as the case requires. Your patient will receive separate baggies for each stage, a care kit, carrying case, removal tool, patient guide, and treatment review plan.



- Made from a premium thermoformed plastic material that is engineered for precision fit, ongoing force retention, and exceptional durability.
- Marked with a unique identifier code for the case and the stage.
- Tear and stain resistant.
- 501(k) cleared



TREATMENT OPTIONS



KEY DENTAL
AESTHETICS

4764 N. Highway 16 | Denver | NC | 28037
keydentalinc.com | 704.489.9978

*Digital cases only. Type ALIGNERS25 in notes section when submitting case. One discount per office. Not to be combined with any other offer or coupon. Coupon Expires: December 31, 2022

AVAILABLE TO DIGITAL CUSTOMERS ONLY

WHAT YOU GIVE...




Scans of lower and upper jaw

Rx with treatment options

Patient photos

WHAT YOU GET...

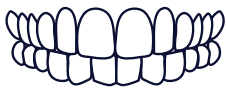

Review case and approve or request design

Editable Clear Aligner Studio 3sz file

Manufacturing Files

PDF of treatment plan

FULL CONTOUR MOVEMENT PHILOSOPHY

	Anterior Movements Case Maximums	Posterior Movements Case Maximums	Movements Per Tray Maximums
Rotation	40°	30°	2°
Inclination	14°	10°	1°
Angulation	14°	5°	1°
Forward/Backward	On the lower arch use the most buccal tooth as a reference point	Distalization 2mm** Mesialization 1.5mm	.25mm
Extrusion	2.5mm	1mm	.25mm
Intrusion	2.5mm	2mm	.25mm
Left/Right	2.5mm	2.5mm**	.25mm

**Especially applicable to upper molars

Each tray is meant to be worn for 2 weeks. Depending on the patient, wear time can be increased or decreased with guidance by the treating doctor.

